► Gehbock «UNIVERSAL» Walking Frame »UNIVERSAL«

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⚠ **Safety Instructions:**

- *The weight of the user must not exceed 110 kg!*
- *Always stand in the middle of the frame and move forward inside the frame. All four feet of the Walking Frame must stand safely on the ground.*
- *Do not lean on the frame on one side, there is a risk of tipping and falling!*
- *Use the product only on flat, solid surfaces. Unevenness, holes or a soft surface can lead to overturning or falls! In case of damp, wet or icy surfaces, there is an increased risk of slipping and falling, adjust to it in your walking behaviour! Use the frame mainly indoors.*
- *Before each use, check the correct seat of the adjustment buttons, they must be fully locked in, all 4 legs must be set to the same height.*
- *Height adjustment, folding or folding apart of the product, increases the risk of clamping!*
- *Do not attach objects, bags, etc. to the Walking Frame »UNIVERSAL«.*
- *Do not use the product as a stand-up or seat aid, do not climb or sit on the cross-struts!*
- *Keep the product away from unattended children*

Warranty

According to the legal warranty claims. Excluded from the guarantee: Damage due to improper use, normal wear and tear as well as defects that have only insignificant effects on the value or serviceability of the device. Technical changes and attachments as well as improper maintenance will void the warranty and product liability in general.

We reserve the right to make changes that are in the interest of technical improvement.

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Valentine International Ltd., 8th Floor, No. 149, Sec 2, Ta Tung Road,
221 Hsichih District, New Taipei City, Taiwan R.O.C.

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EN**Height adjustment:**

- ▶ Push in the adjustment button ①.
- ▶ Pull out the telescopic tube on the desired height.
- ▶ Let the push button snap in again safely.
- ▶ **Caution:** Push-button must be snap in visible and noticeable from the bore in the footpipe.
- ▶ **Important:** All 4 feet must be set to the same height!

EN**Unfold:**

- ▶ Swing both side parts on the handles outwards.
- ▶ The locking system must audibly click into place.
- ▶ Check the folding lock, the side panels must not move.

Fold up:

- ▶ Press the locking button down on the upper crossbar.
- ▶ At the same time, pull the right side part inwards, then the left part.
- ▶ **Caution:** When folding up/unfolding, there is a risk of clamping for the hands!

The product can be used as a reciprocal ① as well as a rigid ② walking frame by moving the holding clamps.

This setting should only be made by qualified personnel and must not be changed by the user without authorization!

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EN**Intended Use:**

The Walking Frame »UNIVERSAL« is a medical device and is designed as a gait training and walking aid, preferably indoors. The product can be used as a rigid as well as a reciprocal walking frame by moving the holding clamps.

Indication:

Restricted walking ability, walking weakness, strong gait insecurity, walking exercises

Contraindication:

severely impaired sense of balance, insufficient gripping ability, loss of limbs and/or joint damage to upper extremities, insufficient physical or mental abilities to use the aid

Versions:

There are different walking aids, the selection should be made after consultation with the doctor / therapist together with the specialist staff. The selection, assembly and instruction in use must only be carried out by authorized, trained specialists. First, try out how to handle the selected walker indoors on a suitable, flat surface with an accompanying person. The product must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user. Any serious incidents arising in connection with this product must be reported to the manufacturer or the competent authority without delay.

Before use / maintenance:

First, read the present instruction manual in its entirety and keep it until the end of the product life. The Walking Frame »UNIVERSAL« is delivered pre-assembled and adjusted by your specialist dealer in height and function type to your needs. Please check if the product is undamaged before first use and in each use. Check the correct seat of the screws and push-buttons, as well as the rubber buffers for wear. In case of signs of damage or wear, the frame must not be used (any more), please contact your dealer in this case. Only original spare parts must be used, only spare rubber buffers with steel inserts can be used!

Cleaning / disinfection:

*Do not use sharp or abrasive utensils or detergents for cleaning. Clean the frame with a soft, damp cloth, if necessary using standard, mild household cleaners. (Please do not use oily or fatty cleaners!) If necessary, the product can be disinfected with a commercially available, chlorine-free disinfectant in the wiping process. **Important:** When using disinfectants, the manufacturer's application requirements must be observed.*